

3B Board

Manual English

 **etac**[®]



Illustrations

Illustration 1

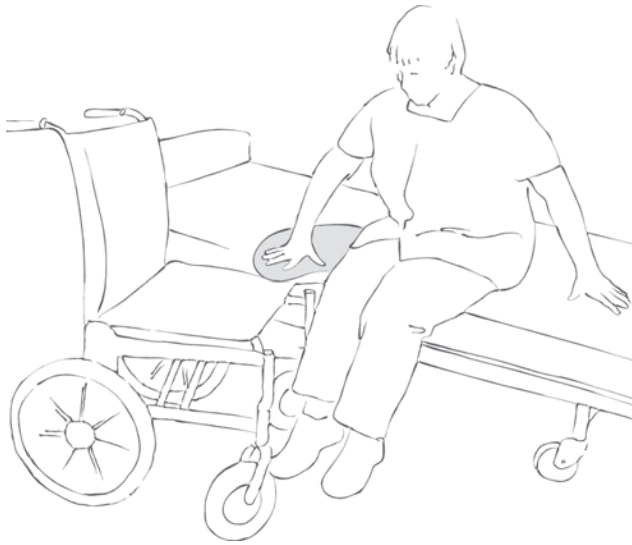


Illustration 2



Illustration 3



Contents

	Heading	Page
1.	General/Features	4
2.	Practical handling	5
3.	Handling/Materials/Cleaning	6
4.	Size/Combinations	7

General

Thank you for choosing an Etac product.

To avoid accidents and injury when moving and handling the products, read the manual carefully.

The person described as "the client" in this user manual is the person who is lying or sitting on the product. The carer or carers are the people who manoeuvre the product.



This symbol appears alongside the text in the manual. It draws the reader's attention to points at which there may be a risk to the health and safety of the client or carer.

The products comply with the standards applicable for Class 1 products in the European Council Directive MDD 93/42/EEC on medical devices.

At Etac we strive to improve our products all the time and therefore we reserve the right to make changes to products without prior warning. All measurements given on illustrations and similar material are for guidance only and Etac cannot be held liable for errors and defects.

The information given in this manual, including recommendations, combinations and sizing, does not apply to special orders and modifications. If the customer makes adjustments, repairs or combinations not predetermined by Etac, the Etac CE certification and Etac warranty will be voided. If in doubt, please contact Etac.

Warranty: Two-year warranty on material and manufacturing defects, provided that the product is used correctly.

For further information on the Etac transfer range, see www.etac.com.

The product can be scrapped in accordance with national regulations.



Washing



Iron



Never leave on the floor



Max. weight



Handwash



Do not dryclean



Warning



The product can be scrapped in accordance with national regulations



Wipe off



Do not bleach



Read the User Manual



Manufacturer



Tumbledry



Risk of sliding down



Batch no./Lot no.



Class 1 - European Council Directive MDD 93/42/EEC on medical devices

Features

The 3B Board can be used to transfer clients from wheelchair to bed, bath/bathchair, car seat, etc., and vice versa.

Use

Positioning

Position the wheelchair at an angle of about 45° to the bed. Position the 3B Board so that the non-slip circles on the backside lie securely against the wheelchair seat and the surface of the bed on the other side. Positioning is easier if the client either leans in the opposite direction or shuffles him/herself sideways to sit on the glide board (Illustration 1)

Use

The client can now slide/"shuffle" slowly over into the wheelchair by pushing himself away from the mattress (Illustration 2).

Depending on the client's resources, the carer can guide the client, supporting the upper body, possibly using a Support Belt, Smartmove or Sling.

Tips

If a slope is established (e.g. if the bed is slightly higher than the wheelchair), the client will slide more easily from one to the other.

In connection with manual handling, it will often be an advantage to use a Pediturn turnplate for the feet and a Support Belt.

To remove the 3B Board, the client moves his/her weight away from the glide board, lifts and then pulls it free (Illustration 3).

Use

Special features

Always check the product before use and after washing.

Never use a defective product. If the product shows signs of wear-and-tear, it must be scrapped.

Read these instructions carefully.

It is important that carers receive instruction in manual handling.

Etac offers advice and training for carers.

For further information, contact Etac.

Always use the correct manual handling techniques.

Remember to use the client's resources.

Remember to apply brakes to lock wheelchair and bed.

The client must have good balance.

There is a risk that the client may slide down.
Never leave the client alone on a glide board or on the edge of the bed.



Never use the 3B Board under wet conditions as the underside of the board may become slippery in contact with water.



Never leave the 3B Board on the floor.

To ensure that the client feels safe and that every manual handling is smooth, always plan the manual handling in advance.

If in any doubt - please contact Etac.

Materials

The 3B Board is made of polyethylene.

Cleaning

The product should be checked regularly, preferably each time it is used.



Never try to repair a product yourself. Never use a defective product.

The 3B Board can be wiped off with soapy water or disinfectant.



Articles

Article no.	Description	Size (cm)
IM403	3B Board	W22.5 x L76.5

Combinations

Article no.	Description	Size (cm)
IM99836GT	Pediturn, turnplate for floor use	Ø40
IM99700	Pediturn, hard turnplate, for floor use	Ø38
IM8017015	Support belt, 11 handles, X-large	W15 x L170
IM8015013	Support belt, 8 handles, large	W13 x L150
IM8013013	Support Belt, 6 handles, medium	W13 x L130
IM8010513	Support Belt, 5 handles, small	W13 x L105
IM8010509	Support Belt, 4 handles, small	W9 x L105
IM428	Sling, 2x4 handles	W20 x L185
IM425	SmartMove, 2 handles	W21 x L60

Etac Sverige AB
Box 203, 334 24 Anderstorp, Sweden
Tel +46 371-58 73 00
Fax +46 371-58 73 90
info@etac.se www.etac.se

Etac AB (export)
Box 203, 334 24 Anderstorp, Sweden
Tel +46 371-58 73 30
Fax +46 371-58 73 90
info@etac.se www.etac.com/exp

Etac AS
Pb 249, 1501 Moss, Norway
Tel +47 815 69 469
Fax +47 69 27 09 11
hovedkontor@etac.no www.etac.no

Etac A/S
Egeskovvej 12, 8700 Horsens, Denmark
Tel +45 79 68 58 33
Fax +45 75 68 58 40
info@etac.dk www.etac.dk

Etac GMBH
Bahnhofstraße 131, 45770 Marl, Germany
Tel +49 236 598 710
Fax +49 236 598 6115
info@etac.de www.etac.de

Etac Holland BV
Fluorietweg 16a, 1812RR Alkmaar, The Netherlands
Tel +31 72 547 04 39
Fax +31 72 547 13 05
info.holland@etac.com www.etac.com

EVOCARE AUSTRALIA PTY LIMITED

A.B.N. 98 078 566 604

Trading as **EVOCARE** and **L&M EQUIPMENT**

P.O. Box 1144, Stafford Qld. 4053

Ph: 07 3355 8000 Fax: 07 3355 5043

Website: <http://www.evocare.com.au>

E-Mail: sales@evocare.com.au

workshop@evocare.com.au



Quality
ISO 9001

etac®
Creating Possibilities