

Centrius Mattress Replacements CARILEX - CEN-5, CEN-7, CEN-8

- Centrius 5 - Place overlay on mattress
- Centrius 7, Centrius 8 - Remove Foam Mattress - Place
- Secure Centrius Overlay to foam mattress with corner elastic straps
- Check CPR connections are pushed in (located at the headend, under mattress cover where red CPR tag is)
- Make sure that mattress/ tubes don't interfere with bed operation.
- Place the Centrius control unit at the bed with the hooks at the rear of the control unit
- Ensure the power cords are connected to the control unit and connected and turned on the wall electrical outlet
- Turn the power on using the rocker switch on the right side of the Centrius Control unit
- Power the Pump by pressing Power/Mute button.
- Control unit is ready when green lights stops flashing.
- It is AUTO SET Pressure - If needed - Adjust pressure manually (Firm, Medium, Soft)
- MAX inflate - maximum pressure in all cells - reverts to alternating after 20 mins
- Dynamic- Alternating Mode - Press Mode button
- Constant Low Pressure - Press Mode Button.
- Panel Locked- press Power/Mute button to inlock pump
- Transport - Press Max inflate - disconnect Connecting tubes and connect into themselves.

Centrius Mattress Replacements CARILEX - CEN-5, CEN-7, CEN-8

Once mattress is inflated, place patient on the mattress, perform bottoming out assessment (place hand under the mattress cover on the deflated cell to check the resident is suspended off the bed base) if mattress needs more air simply increase pressure. DO NOT tuck sheets in too tightly as this could affect the mattress function

ALARMS

- **ALERT!!** Indicates Low Pressure. Please check mattress, make sure the CPR is pushed in, tubes are connected to the control unit, check the resident weight is correct (increase if needed), check air cells are not damaged
- **RESET** - to reset control unit, turn pump on and off using the **POWER/ MUTE**
- **No Power** - **POWER/MUTE** is Red- check power cord connection at the control unit and the wall socket, check control unit is powered on (located on the side of the pump)
- **Bottoming out/ mattress not inflating** - Please check mattress, CPR tag is pushed in, tubes are connected to the control unit, increase pressure if needed, check air cells are not damaged
- If your patient is not receiving pressure relief please contact medical staff/ nurse immediately

CPR

Turn Red CPR tag at the head of the bed for emergency deflation and turn off the power unit by pressing the power/ mute button on the control panel at the same time. To resume normal operation, make sure that CPR is secure, press the power/ mute button again to reset the mattress pressure.