

Entrix 7 Mattress Replacement CARILEX - APO-EN-7

- Remove existing mattress
- Place Entrix 7 Mattress on Bed
- Secure Entrix 7 Mattress to MOVING parts of bed Check CPR connections are pushed in (located under mattress cover where red CPR tag is)
- Make sure that mattress/tubes doesn't interfere with bed operation.
- Attach Cable Management tubes to the press studs located on the right side of the mattress.
- Place the Entrix control unit at the bed with the hooks at the rear of the control unit
- Ensure the power cords are connected to the control unit and connected and turned on the wall electrical outlet (cable management is on the right mattress side for power cord storage)
- Turn the power on using the rocker switch on the right side
- Power the Control Unit by press the green on bottom on the control panel.
- Select patient weight (0-150kg)
- Select alternating mode (10min cycle) or constant low pressure
- Once mattress is inflated, place resident on the mattress and perform bottoming out assessment (place hand under the mattress cover on the deflated cell to check the resident is suspended off the bed base) if mattress needs more air simply increase weight setting. DO NOT tuck sheets in too tightly as this could affect the mattress function

Entrix 7 Mattress Replacement CARILEX - APO-EN-7

ALARMS

- Indicates - low pressure. Please check mattress, make sure the CPR is pushed in, tubes are connected to the control unit, check the resident weight is correct (increase if needed), check air cells are not damaged
- RESET - to reset control unit, turn pump on and off at the control board.
- No Power - check power cord connection at the control unit and the wall socket, check control unit is powered on (located on the side of the pump)
- Bottoming out/mattress not inflating - Please check mattress, CPR tag is pushed in, tubes are connected to the control unit, check the resident weight is correct (increase if needed), check air cells are not damaged
- If your resident is not receiving pressure relief please contact medical staff/nurse immediately

CPR

- Pull Red CPR tag at the head of the bed for emergency deflation
- Turn off the power unit by pressing the **power/mute** button on the control panel at the same time.
- Resume normal operation, make sure that CPR is pushed in, press the **power/mute** button again and reset the patient weight.