DK	GH skiftespor	2
GB/US	GH Switch track	6

Appendix to GH3 manual, Item No 550216

Vers. 2.00

Guldmann™

Varenr.:

553925 – 90° højre

553926 – 90° venstre

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554028 – 90° venstre med coating

553991 – 60° højre

553995 – 60° venstre

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551059 – 60° venstre med coating

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551058 – 60° right with coating

551059 – 60° left with coating

1.00	Purpose and use	7
1.01	Manufacturer	7
1.02	Intended use	7
1.03	Use	7
2.04	Troubleshooting	8
3.00	Classification	9
4.00	EC-Declaration of conformity	9

1.00**Purpose and use**

1.01**Manufacturer**

V. Guldmann A/S
Graham Bells Vej 21-23A
DK-8200 Aarhus N
Tel. + 45 8741 3100
Fax + 45 8741 3131
www.guldmann.com

1.02**Intended use**

The GH switch track is designed for transferring a person weighing less than 250 kg. The GH switch track is a device designed for use with the Guldmann GH rail system. Its purpose is to establish a secure link between two separate single rail systems, connected at an angle of 90°, and which at the same time are at the same level measured from the ceiling.

1.03**Use**

The switch track allows a ceiling hoist to move from one single rail system to another or allows the ceiling hoist to change direction into another single rail system.

The switch track is designed to be operated electrically and it is also designed to allow a run-through without electrical activation. A run-through without electrical operation will not allow the carer to change direction.

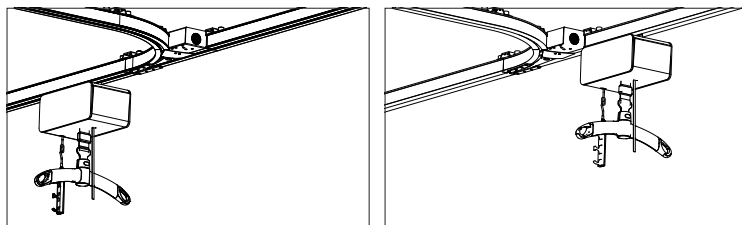
Figure 1. Hoist moving straight ahead in a switch track

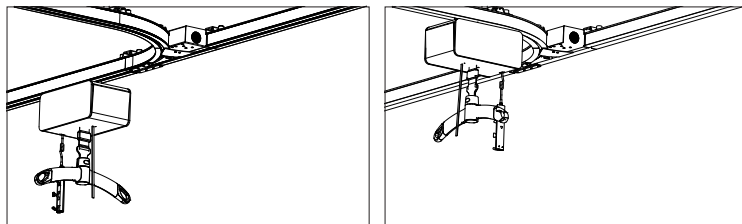
Figure 2. Hoist turning in a switch track

The switch track is operated either by a switch mounted on the wall, a hand control connected to the switch track with a cord or an infrared remote control, which is connected to an infrared receiver inside the switch track.

Switch on wall

The carer must press the button until the switch track is in correct position.

Hand control


The carer must press the button until the switch track is in correct position.

IR-reciever

The carer must press the button until the switch track is in correct position.

For programming and installation of the IR remote control please see:
"User manual "IR remote control Vers. 1.00"

1. One "dot" button is the standard button
2. Other "dot" buttons can be programmed to control the switch track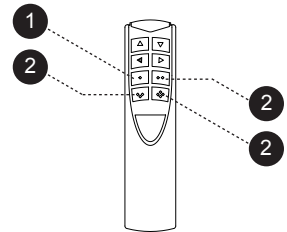

Safety

This product is mechanically protected against derailing and jamming.

The GH switch track must not be used with the GH3 Twin hoist.

2.04

Troubleshooting


The switch track does not respond to the hand control's buttons

1. Check that the switch track has power supply
2. Check that the transformer is switched on and connected to the switch track system
3. Contact the Guldmann Service Team

Degree of protection against harmful ingress of liquids (water)

IP20

Switch track serial number label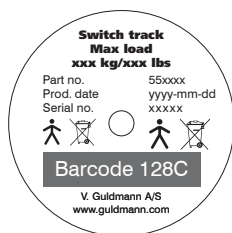

The products are manufactured in compliance with the Council Directive 93/42/EEC of 14 June 1993 – with amendments, as medical device class 1.

| Time to care |

V. Guldmann A/S
Corporate Office:
Graham Bells Vej 21-23A
DK-8200 Aarhus N
Tel. +45 8741 3100
Fax +45 8741 3131
info@guldmann.com
www.guldmann.com

V. Guldmann A/S
Park Allé 382
DK-2625 Vallensbæk
Tlf. +45 8741 3100
Fax +45 8741 3131

Guldmann Inc.
5525 Johns Road
Suite 905
Tampa, FL 33634
Tel. 800 664 8834
Tel. 813 880 0619
Fax 813 880 9558
info@guldmann.net
www.guldmann.net