



Quick Guide

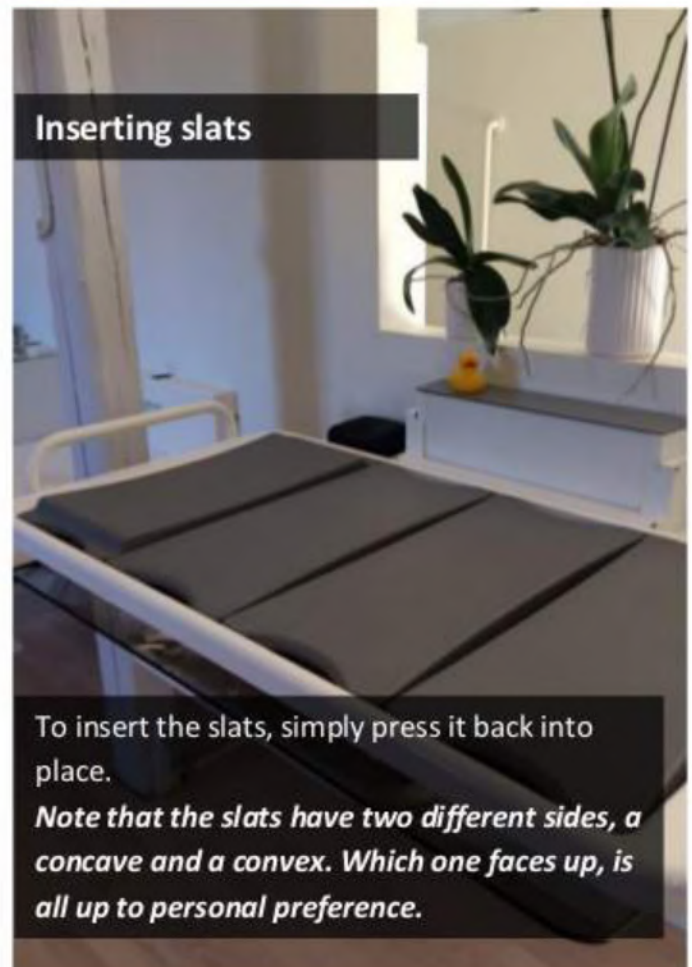

Disengaging the guard rail

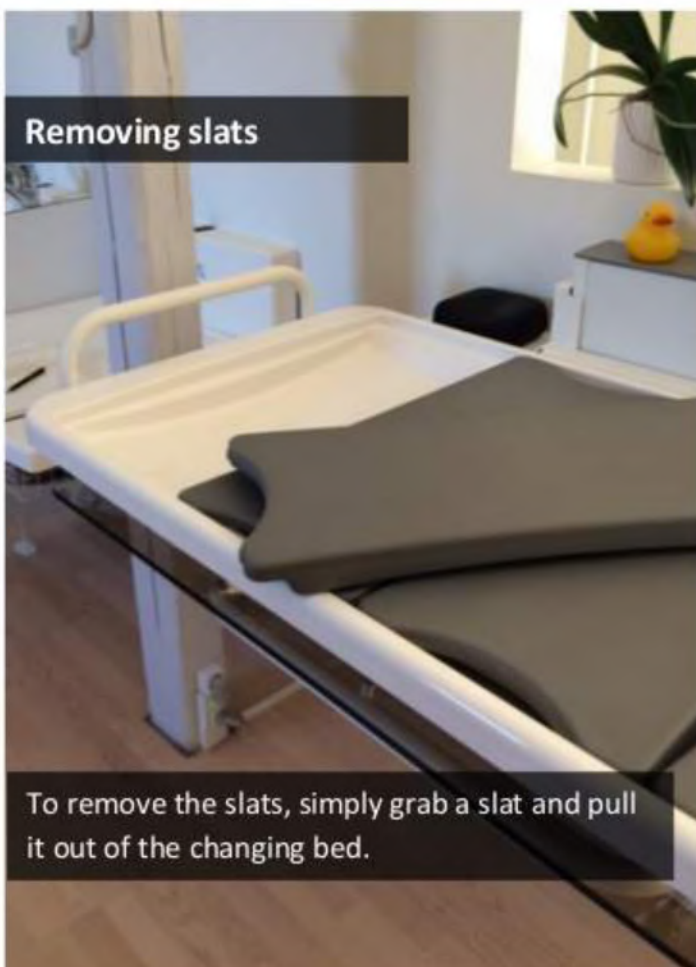

Pulling these two levers upwards unlocks the guard rail, allowing it to be dropped down and pushed down beneath the Shower/Changing bed.
It's locked into place by friction, meaning you don't have to push till you hear it click.



Engaging the guard rail



To engage the guard rail, simply grab it by the white rail and pull it towards yourself and upwards until it snaps into place.

Storing it away



To flip the Shower/changing bed up along the wall, grab it by the edge adjacent to the wall, and pull it up until it snaps into its vertical position.

Remember to disengage the guard rail before doing this!