

TRANSPORT AND MOUNTING TROLLEY FOR VENDLET V5S

VENDLET now offers a trolley for transportation of the VENDLET V5S. The trolley provides better protection of the VENDLET system during transport and simplifies the entire mounting and transport situation.

The transport and mounting trolley was developed in cooperation with Viborg Municipality who wanted to simplify the transport process and streamline the assembly/mounting process. Manager Leo Vejling says: "The transport trolley has really improved our processes; now we can send a single staff member to mount a VENDLET V5S system. At the same time mounting is faster and of higher quality and with optimal Work Health and Safety."

The VENDLET system is suspended from the transport trolley so that the mounting brackets are turned outward. Electronic box, sheets, tools etc., are placed in crates at the bottom of the trolley. It is possible to use your own crates as the frames are adjustable and can be adapted for different sizes.

During mounting the bed height is adjusted so that the bed frame is across from the mounting brackets on the VENDLET V5S system. The trolley is placed alongside the bed frame and as close to it as possible. The VENDLET V5S is then lifted onto the bed frame, first at the head and then at the foot of the bed. The trolley is then moved to the other side where the process is repeated. The mounting brackets are clamped fast, the height adjusted, control box connected and assembly is complete.

To disassemble simply repeat the procedure in reverse.

The trolley is of course fitted with brakes on all 4 wheels as well as a bracket for secure attachment during vehicle transport.

Length: 128 cm
Height: 100 cm
Width: 42 cm

Item no.: 6100300



Figur 1: Transport and mounting trolley



Figur 2: Transport and mounting trolley with 1 VENDLET V5S