

Autoturn

Manual English

 **etac**[®]



Illustrations

Illustration 1




Illustration 2



Illustration 3



Illustration 4

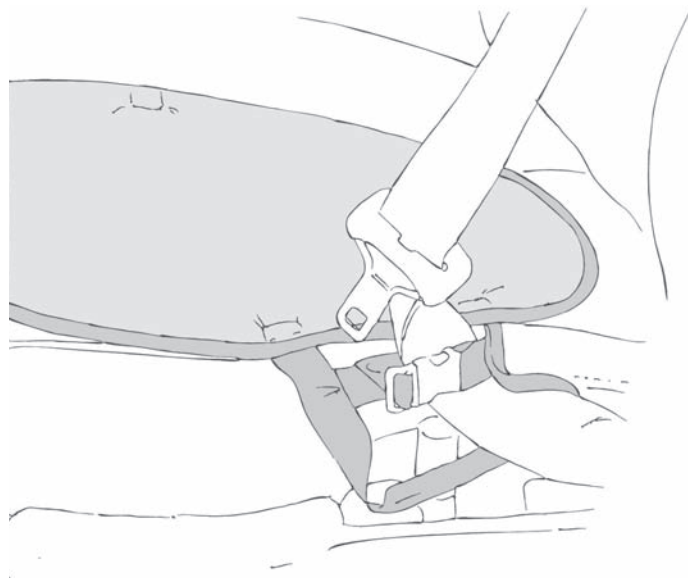

Contents

	Heading	Page
1.	General/Features	4
2.	Practical handling	5
3.	Handling/Materials/Cleaning	6
4.	Size/Combinations	7

General

Thank you for choosing an Etac product.

To avoid accidents and injury when moving and handling the products, read the manual carefully.

The person described as "the client" in this user manual is the person who is lying or sitting on the product. The carer or carers are the people who manoeuvre the product.



This symbol appears alongside the text in the manual. It draws the reader's attention to points at which there may be a risk to the health and safety of the client or carer.

The products comply with the standards applicable for Class 1 products in the European Council Directive MDD 93/42/EEC on medical devices.

At Etac we strive to improve our products all the time and therefore we reserve the right to make changes to products without prior warning. All measurements given on illustrations and similar material are for guidance only and Etac cannot be held liable for errors and defects.

The information given in this manual, including recommendations, combinations and sizing, does not apply to special orders and modifications. If the customer makes adjustments, repairs or combinations not predetermined by Etac, the Etac CE certification and Etac warranty will be voided. If in doubt, please contact Etac.

Warranty: Two-year warranty on material and manufacturing defects, provided that the product is used correctly.

For further information on the Etac transfer range, see www.etac.com.

The product can be scrapped in accordance with national regulations.



Washing



Handwash



Wipe off



Tumbledry



Iron



Do not dryclean



Do not bleach



Risk of sliding down



Never leave on the floor



Warning



Read the User Manual



Batch no./Lot no.



Max. weight



The product can be scrapped in accordance with national regulations



Manufacturer



Class 1 - European Council Directive MDD 93/42/EEC on medical devices

Features

Autoturn is specially developed for use with car seats to make it easy to enter and exit a vehicle.

The two nylon sides facing each other reduce friction during turning.

The down part of the Autoturn pillow has a non-slip surface, to ensure that the device rests firmly on the car seat. Autoturn is fastened to the seat back.

Autoturn features a functional handle which a carer can use to pull the client into place on the car seat.

Use

Positioning

Position the Autoturn centrally on the car seat.
Fit the straps, fasten and tighten securely at seat back.
Fit the functional locking system (next to the functional handle) to the safety belt. When the client turns in the seat, the belt will automatically glide slightly forward.
The safety belt can be secured on the opposite side using the functional locking system (Illustration 4).

Use

The client seats him/herself on the centre of the Autoturn.
The client then raises either both legs simultaneously or one leg at a time inside the vehicle, thus turning the body inside the car (Autoturn is also perfect for turning the client inside the vehicle) (Illustrations 1,2 and 3). This avoids unnecessary strain on the body. The carer can pull on the functional handle to help the client glide into place.
To exit the car, the client swings the legs out of the vehicle, preferably in stages, before raising him/herself to the standing position.

Use

Special features

Always check the product before use and after washing.

Never use a defective product. If the product shows signs of wear-and-tear, it must be scrapped.

Read these instructions carefully.

It is important that carers receive instruction in manual handling.

Etac offers advice and training for carers.
For further information, contact Etac.

Always used the correct manual handling techniques.

Remember to use the client's resources.

For optimal turning effect, make sure that the client is seated in the middle of Autoturn.

Never leave an Autoturn on the floor.



To ensure that the client feels safe and that every manual handling is smooth, always plan the manual handling in advance.

If in any doubt - please contact Etac.

Materials

Top side: Polyester velvet.
Inner surface: 100% nylon.
Backside: 100% non-slip.

Articles

Article no.	Description	Size (cm)
IM99999	Autoturn	Ø45

Cleaning

The product should be checked regularly, preferably each time it is used.

Check that seams and fabric are not damaged.



Never try to repair a product yourself. Never use a defective product.

Washing instructions:

Can be wiped with a damp cloth.



Etac Sverige AB
Box 203, 334 24 Anderstorp, Sweden
Tel +46 371-58 73 00
Fax +46 371-58 73 90
info@etac.se www.etac.se

Etac AB (export)
Box 203, 334 24 Anderstorp, Sweden
Tel +46 371-58 73 30
Fax +46 371-58 73 90
info@etac.se www.etac.com/exp

Etac AS
Pb 249, 1501 Moss, Norway
Tel +47 815 69 469
Fax +47 69 27 09 11
hovedkontor@etac.no www.etac.no

Etac A/S
Egeskovvej 12, 8700 Horsens, Denmark
Tel +45 79 68 58 33
Fax +45 75 68 58 40
info@etac.dk www.etac.dk

Etac GMBH
Bahnhofstraße 131, 45770 Marl, Germany
Tel +49 236 598 710
Fax +49 236 598 6115
info@etac.de www.etac.de

Etac Holland BV
Fluorietweg 16a, 1812RR Alkmaar, The Netherlands
Tel +31 72 547 04 39
Fax +31 72 547 13 05
info.holland@etac.com www.etac.com

EVOCARE AUSTRALIA PTY LIMITED

A.B.N. 98 078 566 604

Trading as **EVOCARE** and **L&M EQUIPMENT**

P.O. Box 1144, Stafford Qld. 4053

Ph: 07 3355 8000 Fax: 07 3355 5043

Website: <http://www.evocare.com.au>

E-Mail: sales@evocare.com.au

workshop@evocare.com.au



Quality
ISO 9001



Creating Possibilities