

Bed String


Manual English

 **etac**[®]



Illustrations

Illustration 1

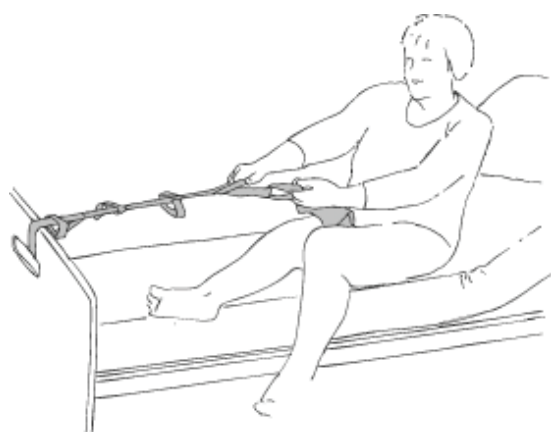

Contents

	Heading	Page
1.	General/Features	4
2.	Practical handling	5
3.	Handling/Materials/Cleaning	6
4.	Size/Combinations	7

General

Thank you for choosing an Etac product.

To avoid accidents and injury when moving and handling the products, read the manual carefully.

The person described as "the client" in this user manual is the person who is lying or sitting on the product. The carer or carers are the people who manoeuvre the product.



This symbol appears alongside the text in the manual. It draws the reader's attention to points at which there may be a risk to the health and safety of the client or carer.

The products comply with the standards applicable for Class 1 products in the European Council Directive MDD 93/42/EEC on medical devices.

At Etac we strive to improve our products all the time and therefore we reserve the right to make changes to products without prior warning. All measurements given on illustrations and similar material are for guidance only and Etac cannot be held liable for errors and defects.

The information given in this manual, including recommendations, combinations and sizing, does not apply to special orders and modifications. If the customer makes adjustments, repairs or combinations not predetermined by Etac, the Etac CE certification and Etac warranty will be voided. If in doubt, please contact Etac.

Warranty: Two-year warranty on material and manufacturing defects, provided that the product is used correctly.

For further information on the Etac transfer range, see www.etac.com.

The product can be scrapped in accordance with national regulations.



Washing



Iron



Never leave on the floor



Max. weight



Handwash



Do not dryclean



Warning



Wipe off



Do not bleach



Read the User Manual



Tumbledry



Risk of sliding down



Batch no./Lot no.



Manufacturer



Class 1 - European Council Directive MDD 93/42/EEC on medical devices

Features

The Bed String is a bed strap with handles sewn alternately on both sides. A self-supportive client can pull him/herself up/down to/from seated and lying position.

Use

Positioning

Fit the Bed String to the bed frame or the foot end of the bed using the loop, so that the Bed String provides the optimal pull and direction for the client.

Use

The client grasps the handles closest to his/her hands when the arms are stretched. The client can pull him/herself up and can change his/her grasp as necessary. The Bed String's functional handles are comfortable and firm to the grasp.

Use

Special features

Always check the product before use and after washing.

Never use a defective product. If the product shows signs of wear-and-tear, it must be scrapped.

Read these instructions carefully.

It is important that carers receive instruction in manual handling.

Etac offers advice and training for carers.

For further information, contact Etac.

Make sure you know exactly how much weight you are trying to move.

Call for extra staff if necessary!

Always use the correct manual handling techniques.

Remember to use the client's own resources.

Never leave the Bed String on the floor.



To ensure that the client feels safe and that every manual handling is smooth, always plan the manual handling in advance.

If in any doubt - please contact Etac.

Materials

100% nylon. 100% cotton.

Cleaning

The product should be checked regularly, preferably each time it is used.

Check that seams and fabric are not damaged.



Never try to repair a product yourself. Never use a defective product.

Washing instructions:

Do not use fabric softener.



Articles

Article no.	Description	Size (cm)
IM429	Bed String with handles	W7.5 x L170
IM991001	Bed String	B5 x L180 cm

Etac Sverige AB
Box 203, 334 24 Anderstorp, Sweden
Tel +46 371-58 73 00
Fax +46 371-58 73 90
info@etac.se www.etac.se

Etac AB (export)
Box 203, 334 24 Anderstorp, Sweden
Tel +46 371-58 73 30
Fax +46 371-58 73 90
info@etac.se www.etac.com/exp

Etac AS
Pb 249, 1501 Moss, Norway
Tel +47 815 69 469
Fax +47 69 27 09 11
hovedkontor@etac.no www.etac.no

Etac A/S
Egeskovvej 12, 8700 Horsens, Denmark
Tel +45 79 68 58 33
Fax +45 75 68 58 40
info@etac.dk www.etac.dk

Etac GMBH
Bahnhofstraße 131, 45770 Marl, Germany
Tel +49 236 598 710
Fax +49 236 598 6115
info@etac.de www.etac.de

Etac Holland BV
Fluorietweg 16a, 1812RR Alkmaar, The Netherlands
Tel +31 72 547 04 39
Fax +31 72 547 13 05
info.holland@etac.com www.etac.com

EVO CARE AUSTRALIA PTY LIMITED

A.B.N. 98 078 566 604

Trading as **EVO CARE** and **L&M EQUIPMENT**

P.O. Box 1144, Stafford Qld. 4053

Ph: 07 3355 8000 Fax: 07 3355 5043

Website: <http://www.evocare.com.au>

E-Mail: sales@evocare.com.au

workshop@evocare.com.au



Quality
ISO 9001



Creating Possibilities