

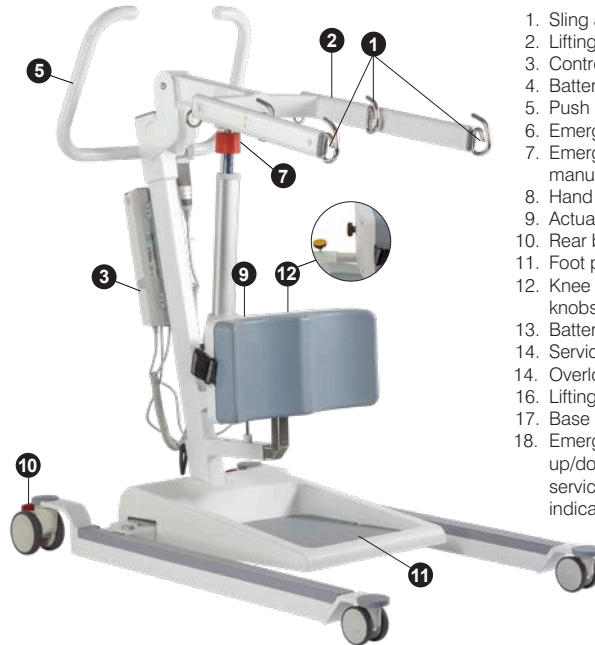


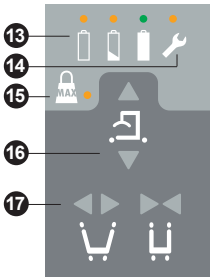

Quick Guide - GLS5.2 Active Lifter

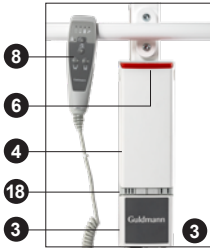

1. Sling attachment hooks
2. Lifting boom
3. Control box
4. Battery
5. Push handle
6. Emergency stop
7. Emergency lowering manual
8. Hand control
9. Actuator
10. Rear braked castor
11. Foot plate
12. Knee pad & adjustment knobs x 2
13. Battery indicator
14. Service indicator
14. Overload indicator
16. Lifting boom up/down
17. Base adjustment
18. Emergency function up/down, battery-, service- and overload indicator

Operators check list before use

- Make sure the lifter is moving freely on its castors.
- Operate the brakes (10) to make sure they are working.
- Lift & lower the lifting boom (2) via the control buttons (16) on the hand control (8).
- Check that the base adjustment (17) is opening and closing, via the hand control (8).
- Check the emergency function up/down by pressing the buttons (17) on the control box (3). Check the manual function by turning the knob (7) in the direction of the arrow.
- Check that the emergency stop button (6) is working. To release the emergency stop, take off battery by pushing buttons on battery sides, and pull battery out and push in place again.

- Release Knee pad adjustment knobs x 2 (12) the pad should move up / down and slide in / out, tighten to lock.
- Make sure footplate (11) is secure and clean.

Charging options

- Charger integrated in control box - plug net cable into power socket.
- When the power is plugged in, the green LED light on the control box lights up.
- The green and yellow diode on the control box (3) illuminates during charging, when fully charged the yellow diode turns off.



Indicator lamps and audio warning

Status	Indicator lamps	Audio signals	Possible GLS5.2 signals			
			Up	Down	Leg spread	Emergency down
Hand control						
All OK	Green		x	x	x	x
Low battery	Yellow		x	x	x	x
Critically low Battery	Yellow	Beeps at button activation	x	x	x	x
Lifter service	Yellow - Is lit during use and 2 minutes after		x	x	x	x
Over load	Yellow - Flashes 3 times + lights up for 10 seconds			x		x
Control box						
Charger	Green + yellow					
Charged	Green					

Accessories


Sling attachment hooks

You have the option to use either the outer hooks or inner hooks. (A risk assessment MUST be carried out before attempting a lift).

Place the sling strap in the hooks. Upon lifting make sure that the strap maintains its position.

Lifting sling

We only recommend using the designed slings for the "Active Lifter." The sling must be checked before every lift – (please see sling user manual for guidance). Label must be intact and readable.

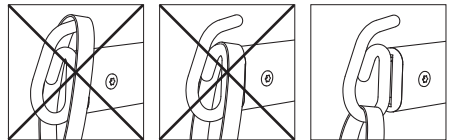

Daily cleaning and maintenance

Clean with a damp cloth and ordinary detergent. Do not use strong acids, bases or alcohol when cleaning. See user manual.



Scan here for user manual, etc. concerning use of the GLS5.2.

Time to care

V. Guldmann A/S

Graham Bells Vej 21-23A, DK-8200 Aarhus N
 Tel. +45 8741 3100
 info@guldmann.com
 www.guldmann.com