

BURMEIER

Ersatzteilliste



Inovia II




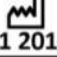



Modellnummer
50.5522.24

Type
240545
240546

BURMEIER available spare parts and accessories are enclosed with the barcode, article, order and serial number. The necessary information can be found on the nameplate, which is attached to the inside of the mattress frame.

BURMEIER GmbH & Co KG Pivitsheider Straße 270 D - 32791 LAGE ☎ + 49 - 05232 - 9841 - 0		
Model	Inovia II 90	
Type	240545	
Serie	36/14	

BURMEIER GmbH & Co KG Pivitsheider Straße 270 D - 32791 LAGE ☎ + 49 - 05232 - 9841 - 0		
Model	Inovia II 100	
Type	240546	
Serie	36/14	





Model Inovia II 90	REF 240545	 51 2015
24 V [~] , max. 5 A, IP X4		
INT. 2 min ON / 18 min OFF		
		

248942


BURMEIER

 Made in EU

 BURMEIER GmbH & Co KG
 DE-32791 Lage

Model Inovia II 100	REF 240546	 51 2015
24 V [~] , max. 5 A, IP X4		
INT. 2 min ON / 18 min OFF		
		

248942


BURMEIER

 Made in EU

 BURMEIER GmbH & Co KG
 DE-32791 Lage




The functional safety and warranty are retained if original BURMEIER spare parts be used!




Burmeier GmbH & Co. KG
 (A company of the Stieglmeyer Group)
Pivitsheider Str. 270 • D - 32791 Lage/Lippe
 Tel. : +49 52 32/98 41- 0 • Fax: +49 52 32/98 41- 41
 Email: service-zentrum@burmeier.de

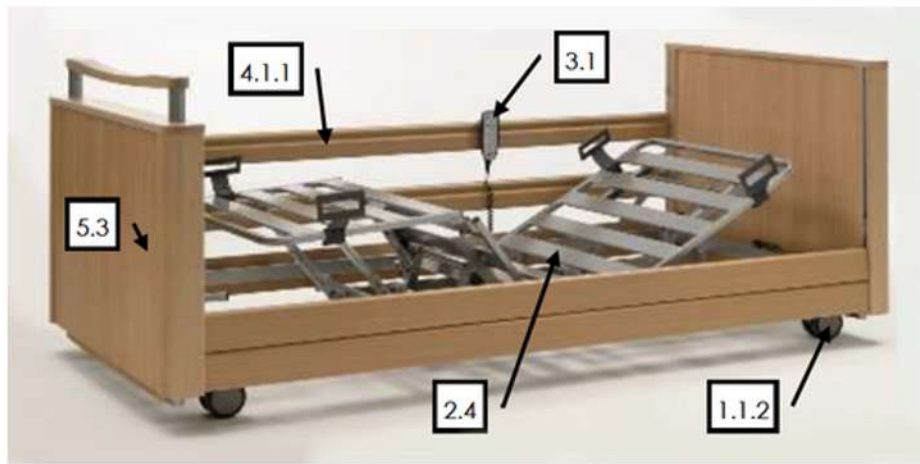

Stand: 01.03.2017

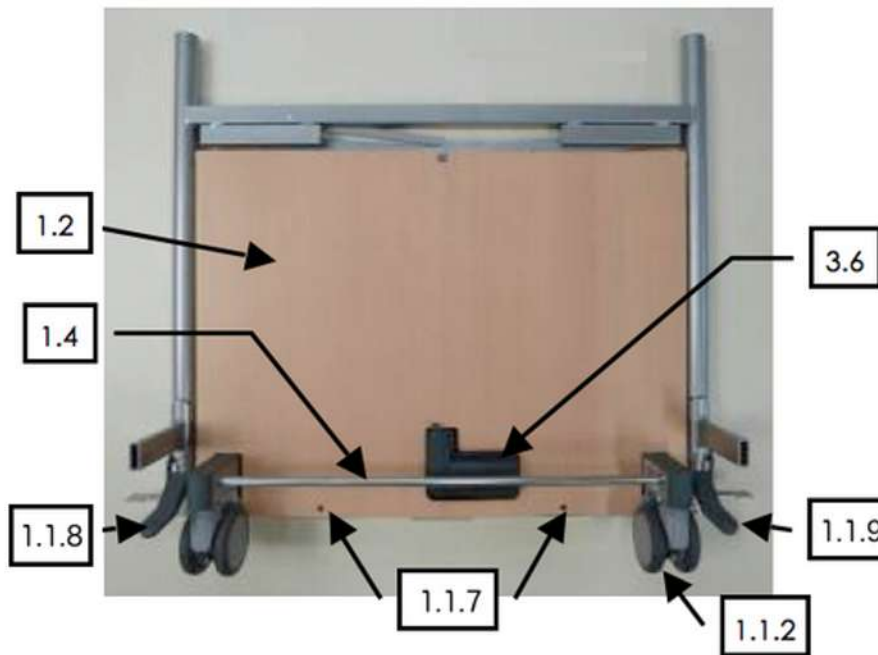

Inovia II Care Bed




1. Chassis Head Section / Foot Section



1.1		chassis complete, (90cm mattress base width) with scissor, motor, plastic covers, brake levers and castors.	192015
1.1.1		chassis complete, (100cm mattress base width) with scissor, motor, plastic covers, brake levers and castors.	200682
1.1.2		double castor Ø 100mm	231343

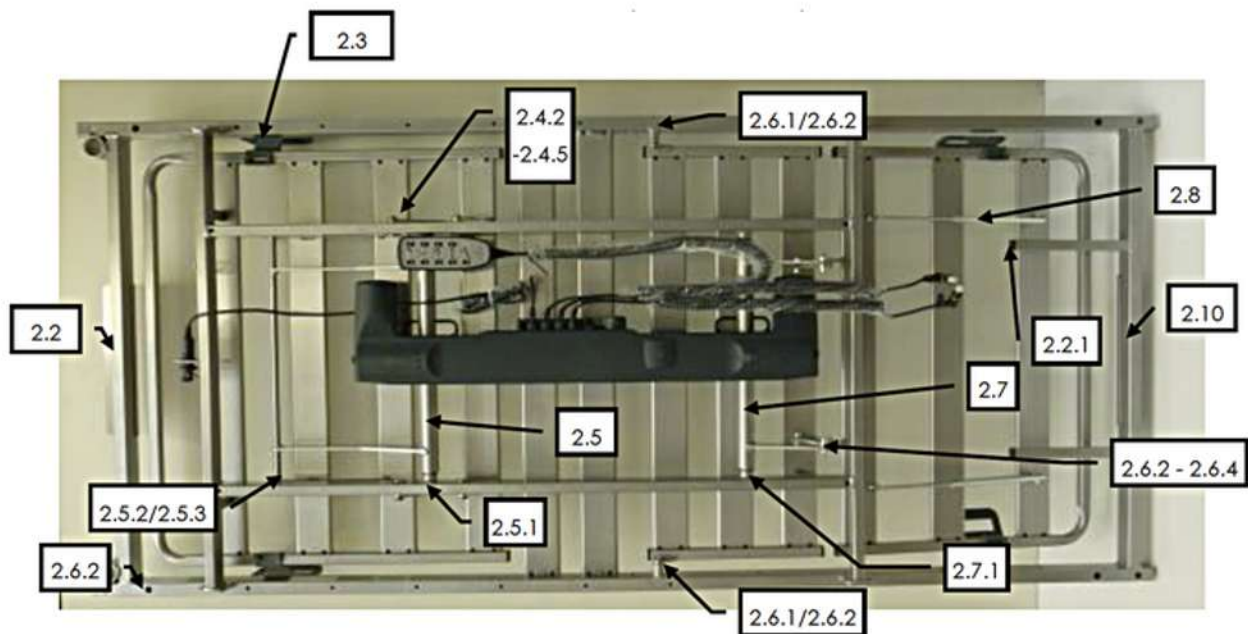

1.1.3		screw ISK M6x8mm (for double castor)	244983
1.1.4		plastic cover (outside) for chassis (90cm)	191973
1.1.5		plastic cover (inside) for chassis (90cm)	191977
		plastic cover (outside) for chassis (100cm)	201940
		plastic cover (inside) for chassis (100cm)	202421
1.1.6		metal stripe for plastic cover (outside)	192841
1.1.7		black dowel, plastic (for plastic cover)	163233
		scissor for chassis (90cm)	
		scissor for chassis (100cm)	
		connecting rod for brake levers (90cm)	
		connecting rod for brake levers (100cm)	


1.1.8		brake lever (left), plastic	194738
1.1.9		brake lever (right), plastic	194739
1.1.10		fixing screw for brake lever	241790
1.1.11		bolt with head for lifting motor \varnothing 12 x 40mm	248240
1.1.12		lock washer (open) \varnothing 12mm	145370
1.1.13		collar stud \varnothing 10x50mm with 18mm head for lifting motor	171801
1.1.14		lock washer (open) \varnothing 10mm	102555




2. Mattress Base


2.1		mattress base 90 x 200cm, complete with metal profiles, including TDC motor	240554
		mattress base 90 x 200cm, complete with metal profiles, including TDC motor, side hinged rails	214815
		mattress base 100 x 200cm, complete with metal profiles, including TDC motor	240555
2.2		metal plate including screws (head end of mattress base)	241803
2.2.1		black square plug, plastic 20 x 20 x 17mm	100105
2.3		mattress holder, plastic	193121
2.3.1		screws for attaching mattress holder	160996

2.4		back rest (inner frame), 90cm with metal profiles, without motor	194802
		back rest (inner frame), 100cm with metal profiles, without motor	241791
2.4.1		Self-adhesive buffer	106738

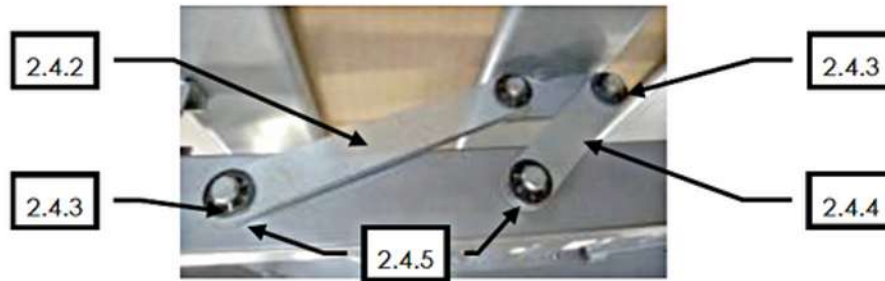


2.4.2		connection piece 148 x 20 x 5mm	208059
2.4.3		lock washer (open) Ø 10mm	102555
2.4.4		connection piece 88 x 20 x 5mm	208060
2.4.5		spacer, plastic Ø 21 / 10, 5 x 2mm	100066
2.5		back rest fork (90cm width)	194801

		back rest fork (100cm width)	241792
2.5.1		lock washer, metal / size 15 DIN6799	167385
2.5.2		black roller for back rest fork, plastic Ø 30 x 13mm	148275
2.5.3		lock washer (open) Ø 10mm	102555
2.6		thigh / lower leg rest (inner frame) 90cm, with metal profiles, without motor	194803
		thigh / lower leg rest (inner frame) 100cm, with metal profiles, without motor	241793

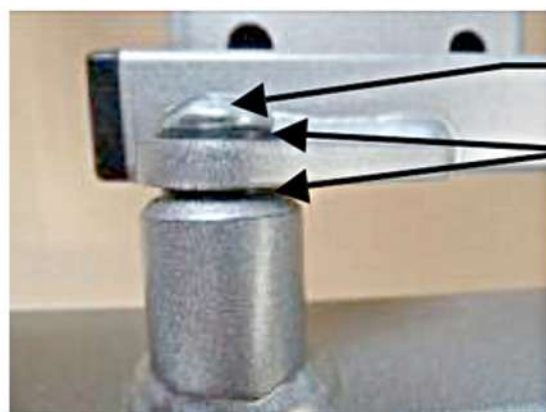

2.6.2
2.6.3

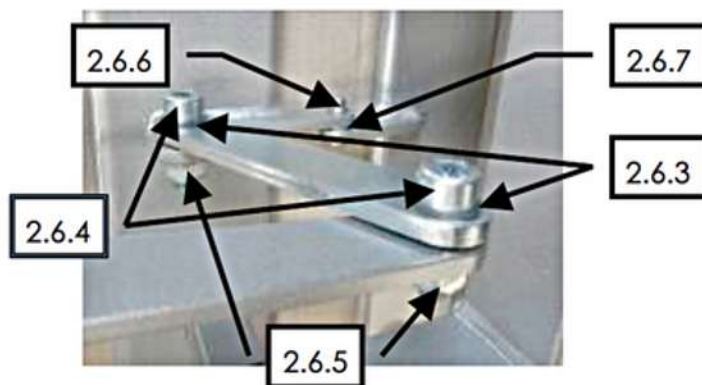

2.6.6
2.6.7
2.6.3
2.6.4
2.6.5

2.6.1		rounded head screw (allen) M8x20mm (to fix thigh rest)	167770
2.6.2		collar socket, plastic, black 15 / 10 / 8x5mm	194804
2.6.3		allen screw M8 x 25mm	101647
2.6.4		self-locking nut M8	115013
2.6.5		blind rivet Ø 6,4x16, 5mm (connection thigh / lower leg rest)	179334
2.6.6		washer, metal Ø 6, 4mm	106043

2.7		thigh rest fork (90cm width)	194740
		thigh rest fork (100cm width)	241791
2.7.1		lock washer, metal size 15 DIN6799	167385
2.8		ratchet, long 215-375 x 30mm	150319
2.8.1		lock washer (open) Ø 8mm (for ratchet)	181858
2.9		self-adhesive white stripe 1000 x 18 x 1mm	248191
2.10		metal plate including screws (foot end of mattress base)	241804


3. Drive System





3.1		hand control with locking function	191491
3.1.1		hook for hand control	192765
3.1.2		locking key for hand control	167882
3.2		Switch-Mode SMPS18 power supply (Linak Nr. 153391)	201938
3.3		cable for lifting motor D-400, helical	187372
3.4		cable from TDC To SMPS18	187375
3.5		Twin drive motor Linak TDC	187373
3.6		lifting motor Linak LA27	702598

4.1 Side Rails

4.1.1		Side rail bar 1955mm x 95.5mm x 28mm	241894
4.1.2		Side rail bar 929mm x 95.5mm x 28mm for VSG	197436
4.1.3		Package hinged side rails (complete for one side) only applicable for lying surface 90cm	228211
4.1.3.1		Swiveling side rail on the left head left, foot right	250745
4.1.3.2		Swivel side rail on the right head right, foot left	250746






4.2 Side Board







4.2.1		Side board 1998mm x 120mm x 16mm for DSG	197399
4.2.2		Side board 1998mm x 120mm x 16mm for VSG	242195
4.2.3		side board 1998 x 120 x 16mm for SSG	229254
4.3		Screw M6 x 35mm	106009
4.4		Washer Ø 6, 4mm	106043
		Package side rails + boards (2 x sides DSG)	241992
		Package side rails + boards (1 x DSG side + 1 x VSG side)	242214
		Side rails guide	214481
		Package side rails spars + boards (2 sides VSG)	242198
4.5		Spacer 10mm for height adjustment	233139






4.6	Assembly package for sliding side rails including post 249582 consisting of:		249582
4.6.1		Ergonomically shaped knob for the center support	191417
4.6.2		Side board cover	709400
4.6.3		Center support for guiding the Lower slide rail 10x10x907mm	713222
4.6.4		Center support for guiding the Upper slide rail 10x10x907mm	713221
4.6.5		Fixed parking element for receiving the side retaining bars at the lowest position	232590
4.6.5.1		Veneer 155.5mm x 45mm x 0.8mm for fixed parking element	232695
4.6.6		Sliding center support with Clamping function and holding Side rails	232585
4.6.7		Lower, clampable sliding element of center support	232586
4.6.8		Upper, fixed sliding element of center support	232587

4.6.9		Stop to limit the travel of the center support	711073
4.6.10		2 x Round head bolt in 6 kt M6 / 70	208061
4.6.11		4 x Round head bolt in 6 kt M5 / 55	236828
4.6.12		1 x Round head bolt in 6 kt M6 / 8	101633
4.6.13		3 x Round head bolt in 6 kt M8 / 25	162351
4.6.14		2 x Round head bolt in 6 kt M8 / 35	197656
4.6.15		2 x Lock washer 8.4	100062
		Assembly Instructions	239699
		Operation Manual	239697

5. Wooden Cover

		Package of wooden covers, Beech headboard, without handle and u. Foot part with handle (for 90cm mattress base) incl. Side rails and mounting kit for side boards	242131
		Package of wooden covers, Beech headboard, without handle and u. Foot part with handle (for 100cm mattress base) incl. Side rails and mounting kit for side boards	240853
5.1		Wooden cover, beech headboard (for 90cm mattress base)	246367
		Wooden cover, beech headboard (for 100cm mattress base)	246369
5.1.2		C-profile, left side head end 20mm x 19.3mm x 749mm x 1.5mm	192958
5.1.3		C-profile, right side head end 20mm x 19.3mm x 749mm x 1.5mm	192960
5.1.4		Spax screw \varnothing 3 x 12mm for fixing C-profile	106088
5.1.5		Spax screw 4.5mm x 45mm for fixing the wooden cover	106107

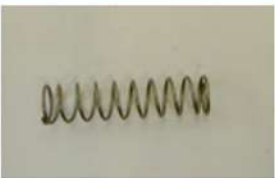5.2		Wooden cover, foot section, beech, with handle (90cm)	246368
		Wooden cover, foot section, beech, with handle (100cm)	246371
5.2.1		C-profile, right side foot end 20mm x 19.3mm x 749mm x 1.5mm	193031
5.2.2		C-profile, left side foot end 20mm x 19.3mm x 749mm x 1.5mm	192959
5.2.3		Beech Handle for wooden cover (90cm) 918mm x 53mm x 30mm	193307
		Beech Handle for wooden cover (100cm) 1018mm x 53mm x 30mm	201370
5.2.3.1		Countersunk fixing screw for handle M8 x 30mm	188159
5.2.3.2		Plastic cover cap for fixing screw, black Ø 18mm	195760

5.3		Release bolt, right 46.5mm x 15mm	700390
5.4		Release bolt, left 46.5mm x 15mm	700389
5.5		Spring for release bolt	178595
		Mounting kit, side rails for a bed, complete	243352
5.6		Mounting kit, side rail guides for one Wooden cover, complete	214481
5.6.1		Metal clip for side rails	700703

6. Lifting Pole

6.1		Lifting Pole, without triangle handle, argentum	164212
6.2		Triangle handle	222225

7. Side Rails for Inovia II with bed extension

		Side Rail 2166mm x 95.5mm x 33 / 33mm	241910
		Side Rail 929mm x 95.5mm x 28mm	197436
		Side Rail 1129mm x 95.5mm x 28mm	197441
		Side Rail 1029mm x 95.5mm x 28mm	247165
		Side Rail 2155mm x 95.5mm x 33 / 33mm	241910

8. Side Boards for Inovia II with bed extension

		Side board right 2198mm x 120mm x 16mm	197418
		Side board left 2198mm x 120mm x 16mm	197422
		Side board right 2198mm x 120mm x 16mm with recess at head side for VSG	242243
		Side board left 2198mm x 120mm x 16mm with recess at head side for VSG	242239
		Side board left 2198mm x 120mm x 16mm with recess in the middle for VSG	247136

		Instructions for use Inovia II / Inovia II 100	241837
--	--	--	--------