

EVO CARE AUSTRALIA PTY LIMITED

A.B.N. 98 078 566 604

Trading as **EVO CARE** and **L&M EQUIPMENT**

P.O. Box 1144, Stafford Qld. 4053

Ph: 07 3355 8000 Fax: 07 3355 5043

Website: <http://www.evocare.com.au>

E-Mail: sales@evocare.com.au

workshop@evocare.com.au



Quality
ISO 9001

KH942 / KH943

KH942E / KH943E

PAEDIATRIC COT



MAIN PICTURE IS KH943 FITTED WITH ALL OPTIONS.
OPERATING, MAINTENANCE & CLEANING MANUAL



n° 89726



A **Hills** Company
KEIM - 015

INDEX

	PAGE
SAFETY PRECAUTIONS	2
OPERATING INSTRUCTIONS	3
MAINTENANCE INSTRUCTIONS	5
CLEANING INSTRUCTIONS	6

SAFETY PRECAUTIONS.

1. The Cots are manufactured and tested to AS2130. To comply a mattress of size (1380mm x 700mm x 50mm or 75mm) must be used.
2. Users must read and follow the manual to ensure all safety precautions are not compromised.
3. For safety precautions place the cot away from curtains, blinds, heaters and power points.
4. For safety precautions keep medication, string, elastic, money, small items and toys out of reach.
5. Serial No., Model No. and Date of Manufacture is located on the underside of the mattress base.
- 6.

OPERATING INSTRUCTIONS

The Mobile Paediatric Cot is designed and manufactured to Australia Standard AS/NZS 2130. It has been designed with safety and function for both child and carer as the utmost importance.

To ensure that all the safety features of the product are not compromised please read the following instruction carefully.

The Paediatric cot is fitted with the following features and functions.

1.1 CASTORS:

For safety the Paediatric Cot is fitted with 2 Ø100mm Total Lock Castors. The Total Lock can be engaged by depressing the outer lever (Fig 1) and disengaged by depressing the inner lever.(Fig 2).



Fig 1



Fig 2

1.2 DROP-END or DROP-SIDE RAILS:

To allow access to either ends or sides of the Paediatric Cot simply lift the rail approx. 10 mm and push the safety catch plate with your knee or other hand. (See Fig. 3 & 4). When the safety catch has been pushed in, lower the rail. When the rails are lifted the catches will self engage.



Fig 3



Fig 4

1.3 TRENDELENBURG OPERATION:

The Paediatric Cot is fitted with an infinitely adjustable Trendelenburg operation. The angle can be set between 0° & 12°. To adjust the base to an inclined angle simply slide the handle out (Fig 6) from under the base and turn in an (anti-clockwise - raise and clockwise - lower) (Fig 5) direction until the desired angle is reached. The handle can then be slid back under the cot out of the way of the carer.

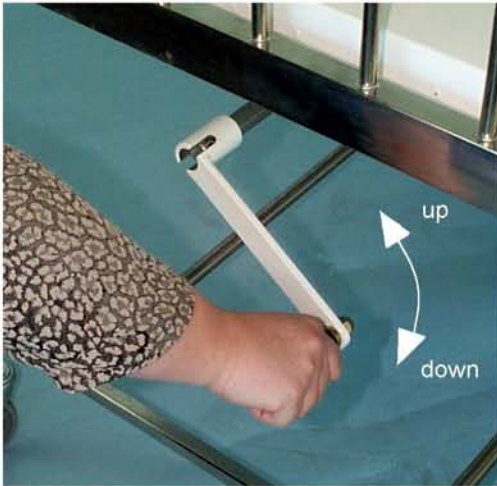


Fig 5

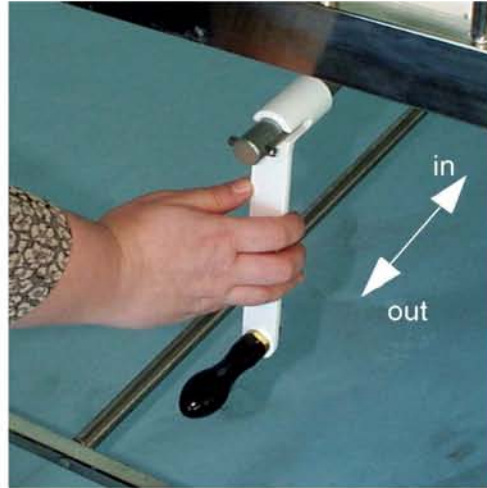


Fig 6

1.4 NO GAP MATTRESS BASE

The Paediatric Cot is fitted with another safety feature. The no gap mattress base is an automatic operation which extends/retracts when the Trendelenburg feature is operated.

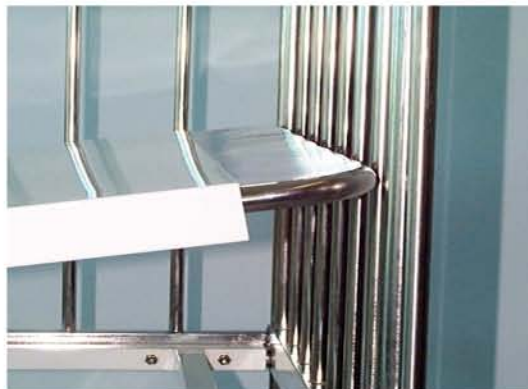


Fig 7

MAINTENANCE INSTRUCTIONS

LUBRICATION

1. CASTORS

The castors are the component most vulnerable to wear and therefore need to be lubricated monthly (Use TAC-2 only). This involves:-

1.1 The Castor head bearing will need lubricating. Simply spray TAC-2 up into the bearing while rotating the castor (Fig 8).

1.2 The wheel also requires lubricating every month. To do this the wheel must be unbolted and removed from the castor frame. TAC-2 can then be sprayed into the hub centre to lubricate the internal bearings (Fig 9).



Fig 8



Fig 9

2. SCREW & NUT

The screw and nut needs lubrication every 2 months. Use a grease gun as per (Fig 10) two pumps of grease only. (Note: Remove all excess grease).



Fig 10

NOTE:1) TAC-2 IS A CRC PRODUCT AND IS AVAILABLE FROM MOST AUTOMOTIVE OUTLETS OR THROUGH:- Kerry Equipment

GENERAL CLEANING

1. The Cot can be cleaned with a low strength detergent, warm water and a soft textured cloth. This includes All stainless steel, Epoxy powder coated framework and the Mattress.

2. MATTRESS CARE

Protect from direct sunlight.
Turn and rotate mattress regularly.
Use mattress protector for best care.

3. SPOT CLEANING

Treat spills and stains as soon as possible.
Gently scrape away any soils or mop away liquid from surface of fabric with damp cloth.
Do not saturate the fabric or interior with water or other cleaning liquid but can be used sparingly. First test on a inconspicuous area if possible.
Do not scrub with stiff brush.
Dry in shade away from direct heat.
Allow to dry thoroughly before re-use.

Distributed by:-

EVOCARE AUSTRALIA PTY LIMITED

A.B.N. 98 078 566 604

Trading as **EVOCARE** and **L&M EQUIPMENT**

P.O. Box 1144, Stafford Qld. 4053

Ph: 07 3355 8000 Fax: 07 3355 5043

Website: <http://www.evocare.com.au>

E-Mail: sales@evocare.com.au

workshop@evocare.com.au



Quality
ISO 9001