



Parts Manual

Oxford® Journey

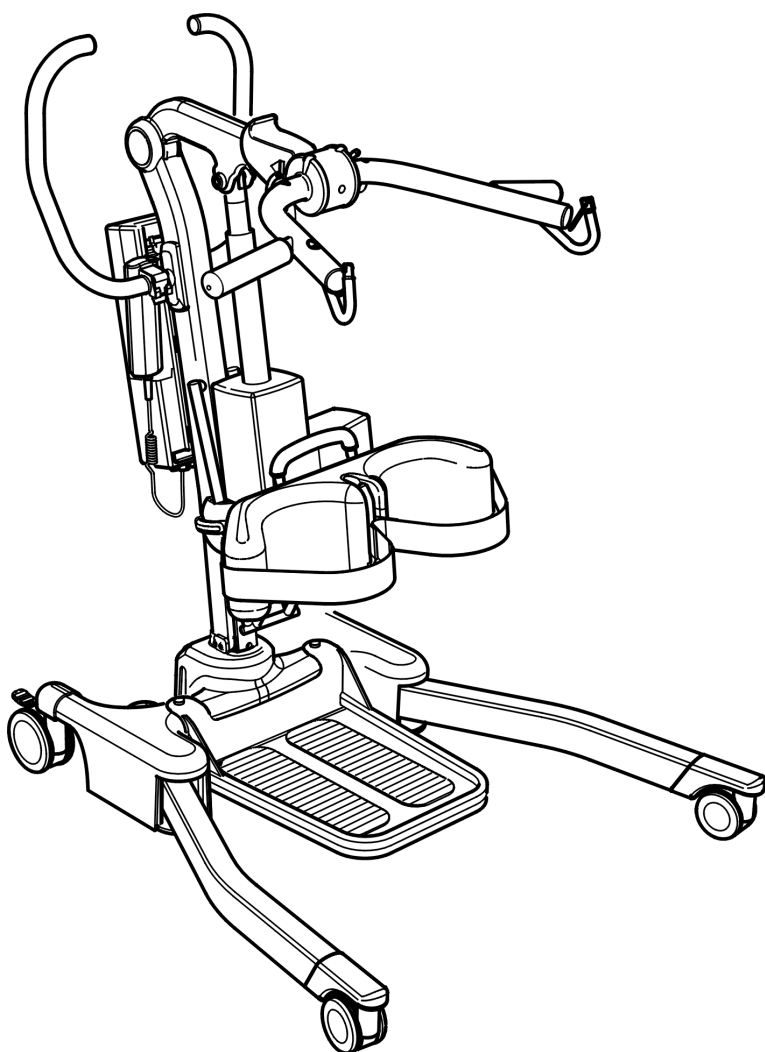

NO	PART NUMBER	DESCRIPTION	QTY
1	0Y0066	100mm rear castor (braked)	2
2	0Y0109	75mm front castor (non-braked)	2
3	0Y0680	Mast assembly (including latch mechanism)	1
4	0Y0664	Cow horn and boom assembly	1
5	0Y0660	Base assembly (inc. base casting, bottom plate, welded post & labels, not internal parts)	1
6	0Y0675	Left leg extrusion (including fixings)	1
7	0Y0676	Right leg extrusion (including fixings)	1
8	39001167	Control box (Smart™ Monitor)	1
	0Y0678	Control box (Standard)	1
9	11012180	Battery pack	1
10	39001202	User handset (Smart™ Monitor only)	1
	0Y0111	User handset (Standard control box)	1
-	39001179	Calibration handset (Smart™ Monitor only) (not shown)	1
-	39001178	Actuator reset handset (Smart™ Monitor only) (not shown)	1
11	39001315	Smart™ Monitor retrofit kit (inc. control box, battery and user handset)	1
12	39001183	Charger and stand assembly (with UK charger lead)	1
13	11012183	Charger lead (UK)	1
14	11012184	Charger lead (EU)	1
15	0Y0677	Actuator - 210mm stroke (including location pins, sleeves, cover and labels)	1
16	0Y0069	Battery and controller bracket (inc. fixings)	1
17	0Y0669	Foot pedal mechanism complete	1
18	0Y0042	Foot push pad assembly complete	2
19	0Y0045	Front castor leg inserts including fixings (plastic)	2
20	0Y0659	Upper fixings kit	1
21	0Y0661	Base fixings kit (all bolts, screws and fixings)	1
22	0Y0684	Push handle - left hand	1
23	0Y0685	Push handle - right hand	1
24	0Y0683	Push handle covers	2
25	0Y0682	Pivot joint plastic cover (includes inner and outer piece)	2
26	0Y0662	Cow horn adjustment slide cover (left and right part)	1
27	0Y0666	Cow horn indicator cover (left and right part)	1
28	0Y0674	Knee pad rod adjustment clamp	1
29	0Y0671	Knee pad adjustment fixings	1
30	0Y0686	Rear base cast cover	2
31	0Y0665	Cow horn handle grips	2
32	0Y0672	Kneepad adjustment handle	2
33	0Y0500	Leg strap	1
34	0Y0499	Knee pad (no fixings included)	1
35	0Y0673	Knee pad bracket (not including kneepad or leg strap)	1
36	0Y0670	Foot tray assembly	1
37	0Y0681	Mast lock pin and handle	1
-	0Y0687	Sling retaining clip (not shown)	10
-	0Y0128	Castor tightening wrench (not shown)	1
-	0Y0668	Decal set (excludes serial number labels) (not shown)	1
-	0Y0070	Handset clip (not shown)	2

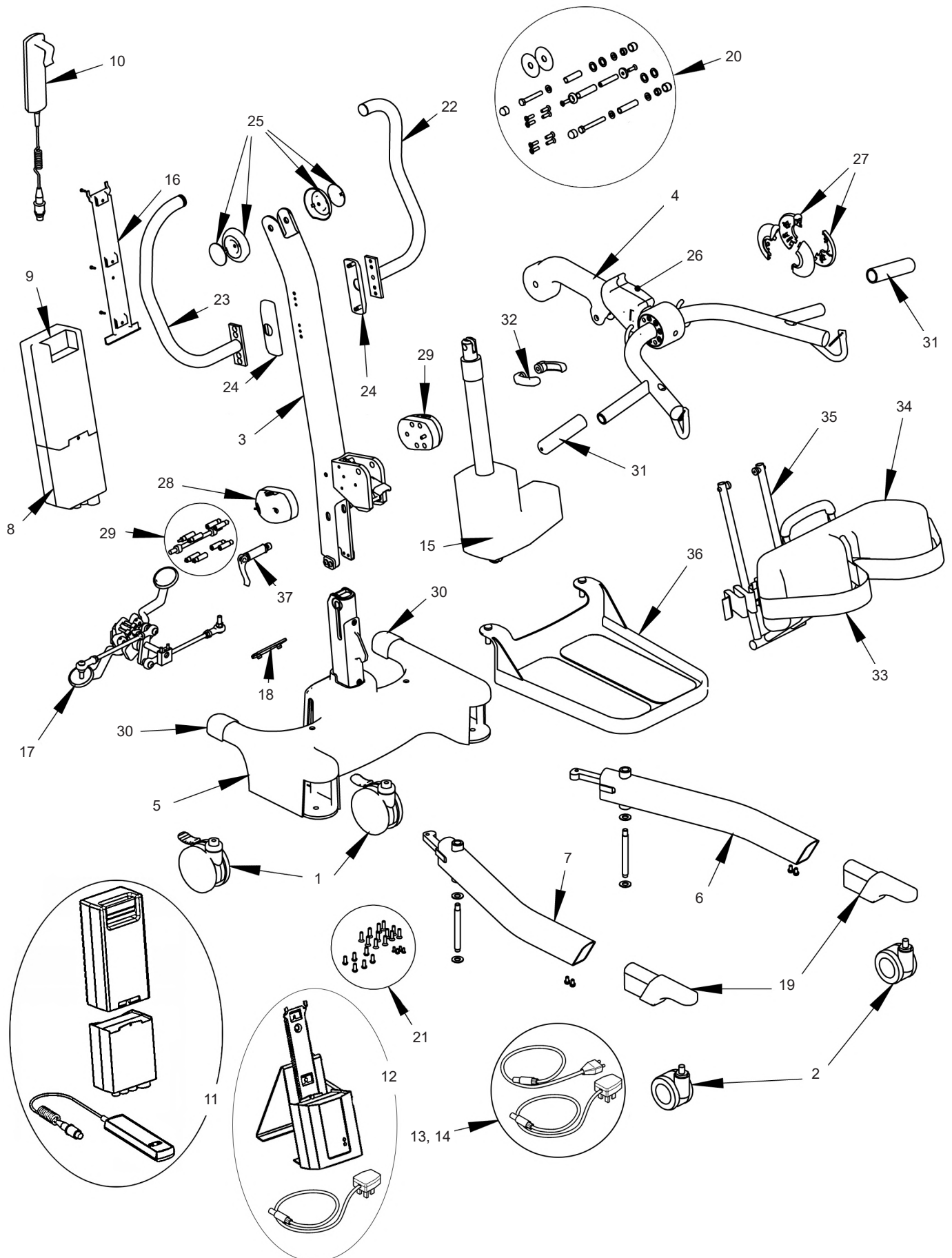

Distributor

EVOCARE AUSTRALIA PTY LIMITED

A.B.N. 98 078 566 604

Trading as **EVOCARE** and **L&M EQUIPMENT**

P.O. Box 1144, Stafford Qld. 4053

Ph: 07 3355 8000 Fax: 07 3355 5043

Website: <http://www.evocare.com.au>

Email: sales@evocare.com.au

workshop@evocare.com.au

warehouse@evocare.com.au

accounts@evocare.com.au



Joerns Healthcare Limited

Drakes Broughton Business Park, Worcester Road, Drakes Broughton
Persore, Worcestershire, WR10 2AG United Kingdom

(T) 0844 811 1156 • (F) 0844 811 1157

www.joerns.co.uk • email: info@joerns.co.uk