

Pediturn

Manual English

 **etac**[®]




Illustrations

Illustration 1



Illustration 2

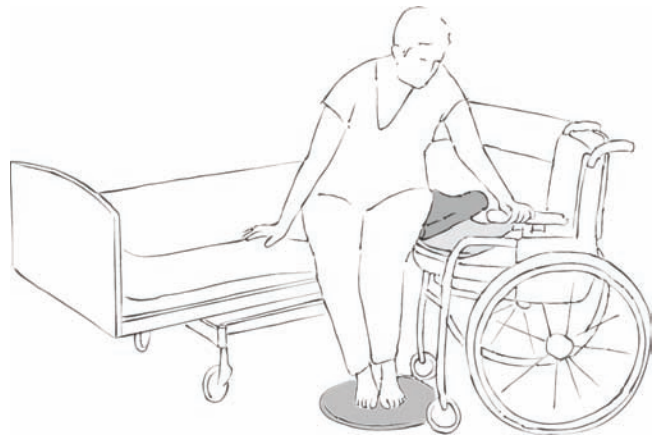

Contents

	Heading	Page
1.	General/Features	4
2.	Practical handling.....	5
3.	Handling/Materials/Cleaning	6
4.	Size/Combinations	7

General

Thank you for choosing an Etac product.

To avoid accidents and injury when moving and handling the products, read the manual carefully.

The person described as "the client" in this user manual is the person who rests his/her feet or stands on the product. The carer or carers are the people who manoeuvre the product.



This symbol appears alongside the text in the manual. It draws the reader's attention to points at which there may be a risk to the health and safety of the client or carer.

The products comply with the standards applicable for Class 1 products in the European Council Directive MDD 93/42/EEC on medical devices.

At Etac we strive to improve our products all the time and therefore we reserve the right to make changes to products without prior warning. All measurements given on illustrations and similar material are for guidance only and Etac cannot be held liable for errors and defects.

The information given in this manual, including recommendations, combinations and sizing, does not apply to special orders and modifications. If the customer makes adjustments, repairs or combinations not predetermined by Etac, the Etac CE certification and Etac warranty will be voided. If in doubt, please contact Etac.

Warranty: Two-year warranty on material and manufacturing defects, provided that the product is used correctly.

For further information on the Etac transfer range, see www.etac.com.

The product can be scrapped in accordance with national regulations.



Washing



Iron



Never leave on the floor



Max. weight



Handwash



Do not dryclean



Warning



The product can be scrapped in accordance with national regulations



Wipe off



Do not bleach



Read the User Manual



Manufacturer



Tumbledry



Risk of sliding down



Batch no./Lot no.



Class 1 - European Council Directive MDD 93/42/EEC on medical devices

Features

Pediturn is a slim, lightweight turnplate for floor use in connection with manual handling in the standing or seated position, where the client must rotate but is not capable of moving his/her legs without assistance.

Use

Positioning

Place Pediturn on the floor. Position the client's feet on the middle of Pediturn so that the client's legs are perpendicular to the device.

Use

When manually handling a client in the seated position, the client's legs will now follow the client, thus avoiding strain on the client's foot, knee and hip joints (Illustration 2).

When manually handling a client standing, it is crucial that the client can stand/bear his/her own weight. The carer guides the client to the standing position and ensures that the client holds his/her balance. Ideally, the carer places one foot on the middle of Pediturn to help the client turn into the correct position. Then the client can slowly sit down again (Illustration 1).

It will often be advantageous to use a Support Belt, Sling or SmartMove with Pediturn.

Use

Special features

Always check the product before use and after washing.

Never use a defective product. If the product shows signs of wear-and-tear, it must be scrapped.

Read these instructions carefully.

Extra care is advised when using Pediturn with spastic clients.

Do not take Pediturn apart – For safety reasons, if the components are taken apart, they may not be put together again.

In such cases, the product must be scrapped.

It is important that carers receive instruction in manual handling. Etac offers advice and training for carers. For further information, contact Etac.

Always use the correct manual handling techniques.

Remember to use the client's resources.

Never leave a Pediturn on the floor after use.



To ensure that the client feels safe and that every manual handling is smooth, always plan the manual handling in advance.

If in any doubt - please contact Etac.

Materials

Top side: Toughtek
Inner surface: 100% nylon.
Backside: Toughtek
Connector disc: synthetic.

Top side: ABS/TPU
Inner surface: HDPE
Backside: ABS/TPU
Connection: Brass and steel

Cleaning

The product should be checked regularly, preferably each time it is used, and especially after washing.

Check that there is no damage to the connection.



Never try to repair a product yourself. Never use a defective product.

Washing instructions:

Can be wiped with a damp cloth.



Articles

Article no.	Description	Size (cm)
IM99836GT	Pediturn turnplate, soft, for floor use	Ø40
IM99700	Pediturn turnplate, hard, for floor use	Ø38

Combinations

Article no.	Description	Size (cm)
IM8017015	Support belt, 11 handles, X-large	W15 x L170
IM8015013	Support belt, 8 handles, large	W13 x L150
IM8013013	Support belt, 6 handles, medium	W13 x L130
IM8010513	Support Belt, 5 handles, small	W13 x L105
IM8010509	Support Belt, 4 handles, small	W9 x L105
IM428	Sling, 2x4 handles	W20 x L185
IM401	Butterfly, glide board	W32 x L66
IM2698121	Swan, glide pad with handles	W45 x L140
IM2698122	Swan, glide board	W25 x L54
IM425	SmartMove, 2 handles	W21 x L60
IM403	3B Board	W22.5 x L76.5
IM404	E-Board	B33 x L60 x H0.5
IM405	E-Board	B33 x L75 x H0.6
IM408	E-Board	B25 x L60 x H0.5
IM410	E-Board	B20 x L45 x H0.5

Etac Sverige AB
Box 203, 334 24 Anderstorp, Sweden
Tel +46 371-58 73 00
Fax +46 371-58 73 90
info@etac.se www.etac.se

Etac AB (export)
Box 203, 334 24 Anderstorp, Sweden
Tel +46 371-58 73 30
Fax +46 371-58 73 90
info@etac.se www.etac.com/exp

Etac AS
Pb 249, 1501 Moss, Norway
Tel +47 815 69 469
Fax +47 69 27 09 11
hovedkontor@etac.no www.etac.no

Etac A/S
Egeskovvej 12, 8700 Horsens, Denmark
Tel +45 79 68 58 33
Fax +45 75 68 58 40
info@etac.dk www.etac.dk

Etac GMBH
Bahnhofstraße 131, 45770 Marl, Germany
Tel +49 236 598 710
Fax +49 236 598 6115
info@etac.de www.etac.de

Etac Holland BV
Fluorietweg 16a, 1812RR Alkmaar, The Netherlands
Tel +31 72 547 04 39
Fax +31 72 547 13 05
info.holland@etac.com www.etac.com

EVOCARE AUSTRALIA PTY LIMITED

A.B.N. 98 078 566 604

Trading as **EVOCARE** and **L&M EQUIPMENT**

P.O. Box 1144, Stafford Qld. 4053

Ph: 07 3355 8000 Fax: 07 3355 5043

Website: <http://www.evocare.com.au>

E-Mail: sales@evocare.com.au

workshop@evocare.com.au



Quality
ISO 9001

etac®

Creating Possibilities