

# SmartMove

Manual English

 **etac**<sup>®</sup>



# Illustrations

Illustration 1



# Contents

	<b>Heading</b>	<b>Page</b>
1.	General/Features . . . . .	4
2.	Practical handling. . . . .	5
3.	Handling/Materials/Cleaning . . . . .	6
4.	Size/Combinations . . . . .	7

# General

Thank you for choosing an Etac product.

To avoid accidents and injury when moving and handling the products, read the manual carefully.

The person described as "the client" in this user manual is the person who is lying or sitting on the product. The carer or carers are the people who manoeuvre the product.



**This symbol appears alongside the text in the manual. It draws the reader's attention to points at which there may be a risk to the health and safety of the client or carer.**

The products comply with the standards applicable for Class 1 products in the European Council Directive MDD 93/42/EEC on medical devices.

At Etac we strive to improve our products all the time and therefore we reserve the right to make changes to products without prior warning. All measurements given on illustrations and similar material are for guidance only and Etac cannot be held liable for errors and defects.

The information given in this manual, including recommendations, combinations and sizing, does not apply to special orders and modifications. If the customer makes adjustments, repairs or combinations not predetermined by Etac, the Etac CE certification and Etac warranty will be voided. If in doubt, please contact Etac.

Warranty: Two-year warranty on material and manufacturing defects, provided that the product is used correctly.

For further information on the Etac transfer range, see [www.etac.com](http://www.etac.com).

The product can be scrapped in accordance with national regulations.



Washing



Iron



Never leave on the floor



Max. weight



Handwash



Do not dryclean



Warning



The product can be scrapped in accordance with national regulations



Wipe off



Do not bleach



Read the User Manual



Manufacturer



Tumbledry



Risk of sliding down



Batch no./Lot no.



Class 1 - European Council Directive MDD 93/42/EEC on medical devices

# Features

The SmartMove can be used to move a client to/from wheelchair/seat, to standing, higher up in the bed, in/out of bed, etc.

# Use

---

## Positioning

---

Position SmartMove to best suit the individual client and where the effect of the device will be greatest.

---

## Use

---

### **Moving the client to standing:**

The carer should make sure that the client is capable of standing. Position the SmartMove either under the small of the back or under the client's rear. Using weight transference, the carer can help the client into the standing position and guide him/her to e.g. another seat (Illustration 1).

### **From seat to seat:**

The SmartMove can also be used as a draw sheet when using a glide board. The client leans away from the direction of movement. The carer positions SmartMove under the client and over the glide board. The client leans in the direction of movement. The carer now pulls on the functional handles.

### **Moving the client higher up in the bed**

Position the SmartMove under the client's rear. Place a glide sheet under the pillow and shoulder blades (if necessary). The carers grasp one handle each and the client glides higher up in the bed.

## Use

### Special features

Always check the product before use and after washing.

Never use a defective product. If the product shows signs of wear-and-tear, it must be scrapped.

Read these instructions carefully.

It is important that carers receive instruction in manual handling.

Etac offers advice and training for carers. For further information, contact Etac.

Always use the correct manual handling techniques

Remember to use the client's resources

There is a risk that the client may slide down. Never leave the client alone on the edge of the bed.



Never leave the SmartMove on the floor.



To ensure that the client feels safe and that every manual handling is smooth, always plan the manual handling in advance.

If in any doubt - please contact Etac.

## Materials

Outer surface: 100% nylon with polyurethane coating

Inner surface: polyester.


Handles: polyester

Press studs: nylon

## Cleaning

The product should be checked regularly, preferably each time it is used, and especially after wash.

Check that there is no damage to seams or fabric.

 **Never try to repair a product yourself. Never use a defective product.**

### Washing instructions:

Can be wiped with soapy water or disinfectant



## Articles

Article no.	Description	Size (cm)
IM425	SmartMove, 2 handles	W21 x L60

## Combinations

Article no.	Description	Size (cm)
IM99836GT	Pediturn, turnplate for floor use	Ø40
IM401	Butterfly, glide board	B32 x L66 cm
IM403	3B Board	B22.5 x L76.5 cm
IM409	Dolphin toilet glide board	L69 cm
IM2698122	Swan glide board	B25 x L54 cm
IM404	E-Board	B60 x L33 x H0.5 cm
IM405	E-Board	L75 x L33 x H0.6 cm
IM408	E-Board	B60 x B25 x H0.5 cm
IM410	E-Board	L45 x B20 x H0.5 cm
IM99836GT	Pediturn	Ø40 cm
IM99700	Pediturn, hard	Ø38 cm

Etac Sverige AB  
Box 203, 334 24 Anderstorp, Sweden  
Tel +46 371-58 73 00  
Fax +46 371-58 73 90  
info@etac.se www.etac.se

Etac AB (export)  
Box 203, 334 24 Anderstorp, Sweden  
Tel +46 371-58 73 30  
Fax +46 371-58 73 90  
info@etac.se www.etac.com/exp

Etac AS  
Pb 249, 1501 Moss, Norway  
Tel +47 815 69 469  
Fax +47 69 27 09 11  
hovedkontor@etac.no www.etac.no

Etac A/S  
Egeskovvej 12, 8700 Horsens, Denmark  
Tel +45 79 68 58 33  
Fax +45 75 68 58 40  
info@etac.dk www.etac.dk

Etac GMBH  
Bahnhofstraße 131, 45770 Marl, Germany  
Tel +49 236 598 710  
Fax +49 236 598 6115  
info@etac.de www.etac.de

Etac Holland BV  
Fluorietweg 16a, 1812RR Alkmaar, The Netherlands  
Tel +31 72 547 04 39  
Fax +31 72 547 13 05  
info.holland@etac.com www.etac.com

## EVOCARE AUSTRALIA PTY LIMITED

A.B.N. 98 078 566 604

Trading as **EVOCARE** and **L&M EQUIPMENT**

P.O. Box 1144, Stafford Qld. 4053

Ph: 07 3355 8000 Fax: 07 3355 5043

Website: <http://www.evocare.com.au>

E-Mail: [sales@evocare.com.au](mailto:sales@evocare.com.au)

[workshop@evocare.com.au](mailto:workshop@evocare.com.au)



Quality  
ISO 9001

**etac**®

Creating Possibilities