


Standard Deluxe Slings

USER INSTRUCTION MANUAL

The Deluxe Standing Sling is an easy fit sling suitable for standing and supporting. It is not a general purpose sling and clients should be assessed as to their suitability. The Deluxe Transport Sling is an easy fit general purpose sling giving full support to most clients. It will not suit double amputees or lifting from a horizontal position.

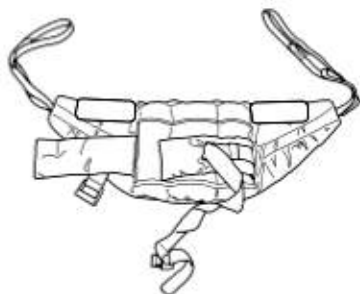

Deluxe Standing Sling



Deluxe Transport Sling



OXFORD RECOMMENDS THE USE OF GENUINE OXFORD PARTS.

Oxford sling and lift products are designed to be compatible with one another. For country specific guidance on sling use and compatibility, please refer to the sling label or contact your local market distributor or Joerns Healthcare.

For the safety of the patient and carer, before using a sling a full risk assessment must be conducted to ensure that the correct sling choice, method of positioning in the sling and procedure for transfer has been determined for the patient.

Check sling and stitching before each use. Using bleached, torn, cut, frayed or broken slings is unsafe and could result in serious injury or death to the patient.

Destroy and discard worn slings. Do not alter slings.

Never leave a patient unattended.



Additional slings are available to suit individual needs.

You are advised that you should always seek the advice of an authorised dealer before purchasing.


We will be pleased to advise you of your nearest authorised dealer.




**Joerns
Healthcare**

ENGLISH

HOW TO FIT THE DELUXE STANDING SLING

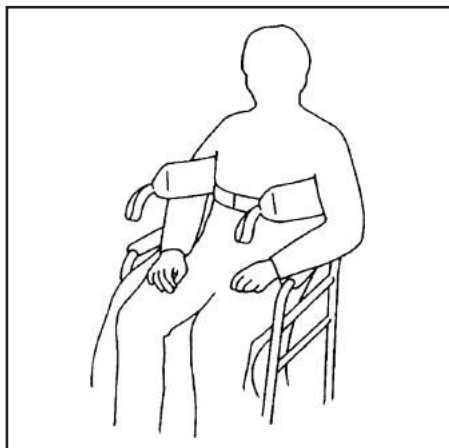

Place sling down behind client's back between base of shoulder blades and bottom of rib cage. Attach strap around the front of the client. This strap need not be tight. It is there to hold the sling in position whilst attaching the sling to the Standaid. Ensure that the client's arms are outside the sling.

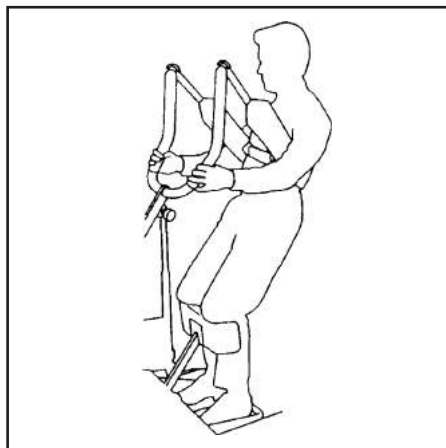

Position Standaid so that the client's feet fit in the foot trays and knees rest against the knee pad. Initially attach the longer loops of the sling to the white hooks at the end of the hand grips. The client's hands should be placed on the black hand grips or support handles. (Once the client is used to the lift, the shorter loops may be used to achieve a more upright position). Then push the up button on hand control to raise the client to the standing position.

NOTE: THIS SLING IS NOT A GENERAL-PURPOSE SLING AND WILL NOT BE SUITABLE FOR ALL CLIENTS.


TIPS FOR USE:

CLIENT SHOULD BE ASKED TO LEAN BACK INTO THE SLING FOR THE DURATION OF THE LIFT.


TALLER CLIENTS MAY NEED TO USE SHORTER LOOPS ON THE SLING FROM THE START.

SHOULD THE CLIENT START TO LOSE TRUNK CONTROL YOU WILL FIND THAT THE SLING WILL SLIP UP BENEATH THE ARMPITS MAKING THE LIFT UNSAFE. IN THIS SITUATION THE TRANSPORT SLING CAN BE USED PROVIDING A RISK ASSESSMENT IS COMPLETED.


HOW TO FIT THE DELUXE TRANSPORT SLING

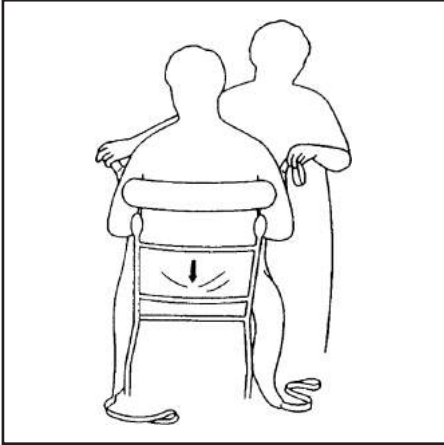


Feed the sling down the back of the client leaving the top of the commode aperture at the base of the spine. Check the sling is square across the client's base.

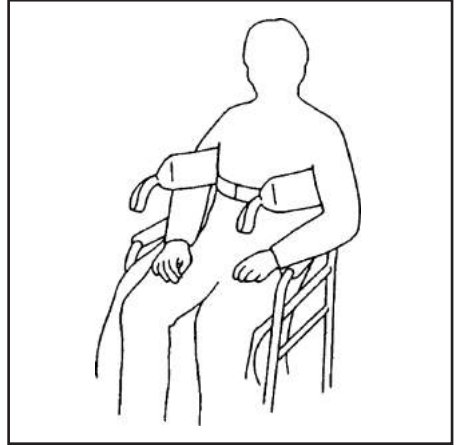

Attach the strap around the chest of the client. This strap need not be tight, it is there to hold the sling in position whilst attaching the sling to the Standard. Ensure client's arms are on the outside of the sling.

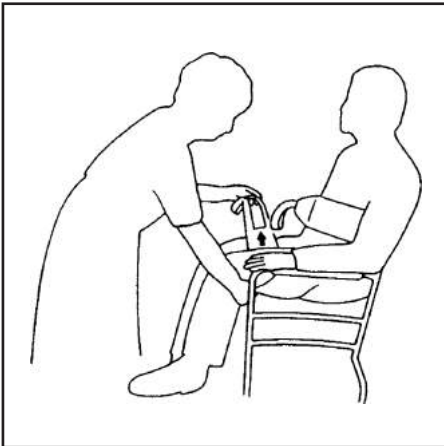

Raise the client's leg and feed the leg strap under, and then up between the legs. Ensure that the sling is not twisted or creased under the thigh. Repeat this procedure for the other leg. Attach the loops on the top of the sling to the white hooks at end of hand grips.

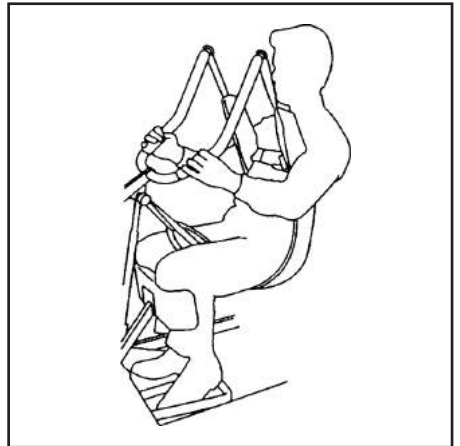

Without crossing loops on the leg straps, attach to left and right black knobs on the lifting fork(s). You would normally use the middle of the three loops. You are then ready to raise the client from the seat and transport. Once clear of the seat, the client will be more comfortable if lowered back to sitting position.


NOTE: ENSURE THAT YOUR CLIENT'S HEAD IS SUPPORTED AT ALL TIMES.

TECHNICAL SPECIFICATIONS

STANDARD	BS EN ISO 10535
SAFE WORKING LOAD	440LBS / 200KG

SIZING

Each sling is marked with the following colour coded symbols, four of which will be crossed. Your sling is the uncrossed size, i.e. in this example Small (Red).

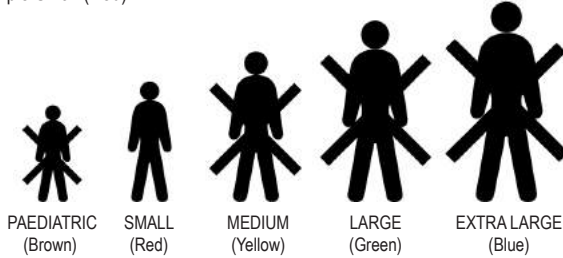

WASHING INSTRUCTIONS

Each sling is marked with the following washing instructions:

DO NOT WASH WITH BLEACH

Bleach will damage material. Slings can suffer damage during washing and drying. Check sling before each use. Bleached, torn, cut, frayed or broken slings are unsafe and could result in serious injury or death to patient. Destroy and discard worn slings. Do not alter slings. Machine wash warm. Cool tumble dry, air dry or dry at very low temperature.



SIZE	Standing	Transport
PEDIATRIC (BROWN)	SPECIAL (Polyester)	SPECIAL (Polyester)
SMALL (RED)	SL13499N (Polyester)	SL13502N (Polyester)
MEDIUM (YELLOW)	SL13500N (Polyester)	SL13501N (Polyester)
LARGE (GREEN)	SL13498N (Polyester)	SL13503N (Polyester)
EXTRA LARGE (BLUE)	SPECIAL (Polyester)	SPECIAL (Polyester)

THESE SLINGS ARE SUITABLE FOR USE WITH THE OXFORD STANDAID



Joerns Healthcare Limited
 Drakes Broughton Business Park, Worcester Road
 Drakes Broughton, Pershore, Worcestershire
 WR10 2AG United Kingdom
 Tel: 0844 811 1156 • Fax: 0844 811 1157
 info@joerns.co.uk • www.joerns.co.uk

