

EVOCARE CONTACT:

Sales: 07 3355 8000

Fax: 07 3355 5043

Email: sales@evocare.com.au

PO Box 1144, Stafford

Q 4053 Australia

Vendlet on Allura Mighty

Call or email for more information

More information on our website:

<http://www.evocare.com.au/vendlet.php>

<http://www.evocare.com.au/allura-care-bed.php>

<http://www.evocare.com.au/bed-dialysis-scales-seca-984.php>

Evocare
Case Study
Bariatric Patient
195kg



The Equipment (see next page)

6000000/6000021	Vendlet V5 (200kg) / Vendlet V5+ (400kg)
3601002	Vendlet Slide Sheet
2600005	Vendlet Turning Sheet
EV94B4AAM	Allura Mighty bed
EV94B4AAMOTC	Timber cover for footboard (optional)
EV94B4AAMOTH	Timber cover for headboard (optional)
EV9743	Mattress
Seca	Bed Scales

The Ideal Combination for handling bariatric clients in bed

After assembly of the Vendlet on the Allura Mighty Bed and positioning the bed on the weigh scale, a floor hoist was used to transfer the client from her old bed onto the Allura Mighty and she was weighed immediately on the Bed Weigh Scale; the result being 195kg. The client's carers were shown how to turn her onto her left and right side for the purposes of; toileting, hygiene and fitting and removal of slings (both sides of the Vendlet were up for her safety; the client requested it). The turn on her left side was a little more difficult as her right leg could not be moved very far over the left leg for stability and an additional carer was required. However, once on her side, the carers offered her gentle support and used the Vendlet to move her closer to the side roller and she is able to grab onto it for support. Wedge cushions were suggested as an aid to this turn and this was noted by the physiotherapist present.

The turn onto her right hand side was much easier because her left leg could shift across and a carer could assist her on my own. Carers were now able to reposition the client in bed, for when she slides down the bed and when a sheet is soiled and requires changing while the client is in bed. The client was apprehensive at first but found the turning gentle and safe. Also, the bed is a good width (1200mm) for her and the mattress is comfortable. The carers are happy to have finally found a system that works and will make their job easier. The carers also liked the look and the quality of the Allura Mighty Bed.

The client was delighted with the ability to profile the bed into different positions, especially into the chair position.



Our catalogue is our comprehensive and up to date website: <http://www.evocare.com.au/>

Specialist Equipment Sales & Service

EVOCARE CONTACT:

Sales: 07 3355 8000

Fax: 07 3355 5043

Email: sales@evocare.com.au

PO Box 1144, Stafford

Q 4053 Australia

Vendlet on Allura Mighty

Call or email for more information

*The ideal equipment
for handling
bariatric people*



**BURMEIER ALLURA MIGHTY 4 SECTION
ELECTRIC NURSING HOME BED**

- Adjustable Back Support and Upper Leg Rest
- Safe working load: **250kg**
- Mattress base - steel grating
- Selectively locking handset with auto-contour and robust construction
- High-quality, rugged Tente castors
- Wooden surround with integrated side frames in beech finish, optional timber cover for head and foot boards
- Linak drive system with 24 Volt technology, live 240 Volt components are outside the bed
- EV9743 (2000 x 1200 x 180) Maxi Pressure Care Mattress



**VENDLET – EASY, GENTLE, SAFE
HANDLING OF PATIENTS IN BED**

Ease of Daily Care Routines

VENDLET makes repositioning of patients in bed an easy one person job at the touch of a button. There will be improved care giver productivity and no more manual handling.

A Comprehensive View of the Patient's Needs

The uniform and gentle procedure ensures patient comfort and a positive contact between care giver and patient. The care giver has a comprehensive view of the procedure making it possible to support the patient's needs during the procedure.

Improved Care Giver Productivity

VENDLET simplifies planning of the daily care routines because the patient in many situations can be handled by one person.

SECA 984 BED SCALES WITH TROLLEY



Always in use: the seca 984 facilitates the easy, gentle and precise weighing of bedridden patients and is an indispensable aid in dialysis and intensive care.

With the integrated lifts, four load cells can be placed easily and effortlessly under the castors of the bed.



Our catalogue is our comprehensive and up to date website: <http://www.evocare.com.au/>

Specialist Equipment Sales & Service