

EVO CARE CONTACT:

Sales: 07 3355 8000

Fax: 07 3355 5043

Email: sales@evocare.com.au


PO Box 1144, Stafford

Q 4053 Australia

How to Use a Frame

Call or email for more information

These frames are designed to be picked up and put down with each step.

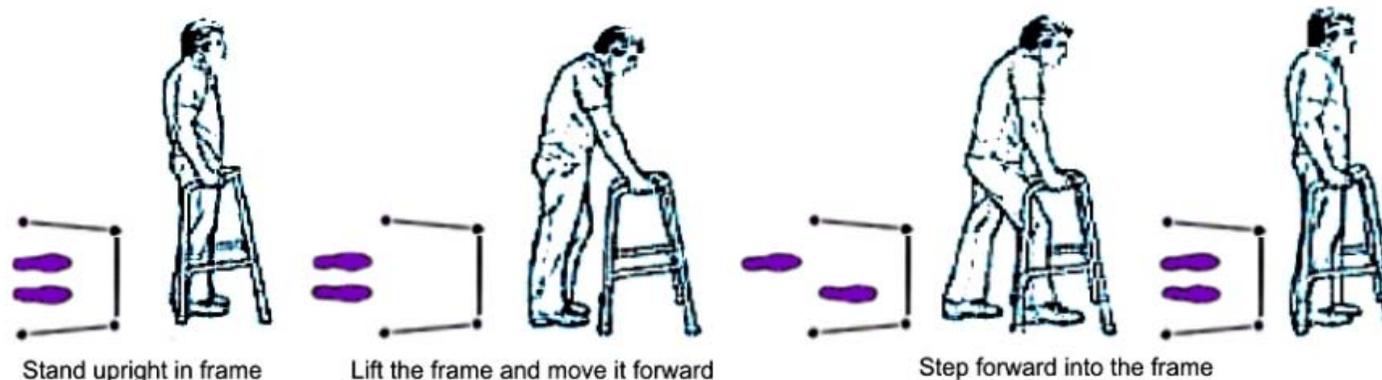

Instructions for Use

- Ensure walker is fully opened and locked.
- Do not step fully into the frame and do not put the frame too far in front.
- Take ramps cautiously and slowly.
- Frame can only be used on steps if all 4 legs fit on the step; place frame up on step and step into it off strongest leg. When going down, step into frame with weakest leg.

Safety

- If the frame is unsteady, make sure it is fully unfolded and locked.
- Always make sure all 4 legs are on the ground before stepping into frame.
- Always move slowly and cautiously.
- Frame is not to be used to assist to sit and stand.
- Seek professional advice to adjust or change frame.
- Be aware of frames weight tolerance.

Maintenance and Care

Servicing is required if:

- Handgrips are worn
- Stopper feet are worn
- Frame is difficult to fold
- Frame is corroded
- Frame parts are loose

Always dry frame if wet.
(not for use when showering)

 **Evocare**[®]

Our catalogue is our comprehensive and up to date website: <http://www.evocare.com.au/>

Specialist Equipment Sales & Service